



# Debrisoft® Lolly

for debridement

## **en** Instructions for use

### **Product description / product composition:**

The head of the Debrisoft® Lolly is composed of monofilament polyester fibres and seamed with a blue, X-ray detectable polypropylene thread (containing barium sulphate and polyester). The interior of the head is coated with polyacrylate. The head is sewn onto a polypropylene handle.

The product is individually packed and sterile (EO sterilization).

Debrisoft® Lolly is intended for single use only. The Debrisoft® Lolly must not be resterilized once the packaging has been opened or reprocessed after initial use. The products are sterile unless the packaging has been opened or damaged.

### **Indications:**

The Debrisoft® Lolly is intended for the debridement of deep – including surgically invasive – to superficial wounds for wound bed preparation.

E.g.:

- diabetic ulcers
- arterial and venous ulcers
- pressure ulcers
- postoperative wounds healing by secondary intention
- trauma, burns/scalds

The Debrisoft® Lolly is used to absorb exudate, debris and keratoses during debridement.

Note: Debridement with the Debrisoft® Lolly is a procedure which can be performed quickly, easily and virtually painlessly and which protects vital tissue.

### **Contraindications:**

None known to date

### **Adverse reactions:**

In very rare cases, skin irritation or allergies may develop.

### **Instructions for use:**

1. Remove the old dressing if necessary.
2. Perform the debridement:
  - a. Soak the head of the Debrisoft® Lolly in saline solution, for example, in accordance with the national guidelines (recommended volume 5-15 ml).
  - b. Debride the wound and/or hard-to-reach areas by applying gentle pressure with the soft head of the moistened Debrisoft® Lolly.
  - c. If necessary, clean the skin around the wound with another moistened Debrisoft® dressing pad.
3. Following debridement, continue with the standard treatment.

### **Precautionary measures:**

- Do not leave the Debrisoft® Lolly in the wound.
- Do not use this product in patients with a known intolerance of or allergy to one or more of the components.



- Never introduce it into wounds with an opening smaller than the size of the head of the Debrisoft® Lolly.
- The head of the Debrisoft® Lolly must not be cut to size or detached from the handle.
- Avoid exerting too much pressure as this could cause the patient pain or result in tissue damage.
- Use on fistulae with unclear connections (e.g. abdomen) must be performed with particular caution.
- It must be noted that under certain circumstances it may not be possible to reach branches/nooks.



Lohmann & Rauscher International GmbH & Co. KG  
Westerwaldstraße 4, 56579 Rengsdorf, Germany  
[www.Lohmann-Rauscher.com](http://www.Lohmann-Rauscher.com)

Packaging Version **4029155/2017-08**